

Early Cancer Screening Test

Patient Preparation:

Component	Special Notations
Fasting/Special Diets	No fasting is required.
Specimen Collection and/or Timing	Routine

Specimen Requirements:

Specimen Type	Collection Tubes	Amount & Quality	Tube Label (print)
Whole Blood	Streck BCT Collection Tube or Nanograb Tube Do not use purple top tube !!!	1-2 tubes; fill tube completely (10 ml) Stable for 7 days at ambient (18°-26°C) temperature* Do not refrigerate or freeze blood samples	Name and Date of Birth Collection Date

* Samples received that were drawn more than 4 days before receipt are unacceptable

Shipping Instruction:

Specimen Type	Packaging Requirement	Shipping Conditions
Whole Blood	<ol style="list-style-type: none"> Place the Streck tube in 50mL conical tube and close tightly. Place tube in a biohazard bag. Place biohazard bag into yellow envelope for shipping. Write sender name and address in upper left hand corner of envelope. 	Ship ambient (18°-26°C) by USPS (pre-paid stamp provided on envelope). Do not heat or freeze.

Ship specimen with Signed Test Requisition Form to:

**Breakthrough Genomics
2 Hughes, Suite 100
Irvine, CA 92618**

**Please
Call: 949-229-0094 or
Email: test@btgenomics.com
for any questions**